




PROTECTIVE COLLAR



DIRECTIONS:

PLEASE READ BEFORE USING THE PROTECTIVE INFLATABLE COLLAR

1. Inflate by pinching and blowing into the retractable air valve. Press and close after filling the inner tube with air.
2. Remove your dog's collar and run it through the inner ring straps of the protective inflatable collar. Make sure the security strap is on the bottom side of the protective inflatable collar.
3. Reattach the collar around your dog's neck. Adjust so that the velcro tab and security strap are behind your dog's head. The security strap will be on the bottom side of the protective inflatable collar.
4. Secure the velcro tab so both ends of the protective inflatable collar are together. Note: Velcro will require moderate strength to pull apart.
5. Disconnect the security strap and wrap one end around your dog's chest behind your dog's front legs. Adjust for fit and reconnect.

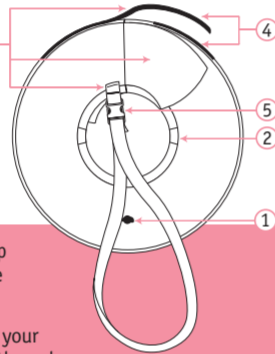

CAUTION: The protective inflatable collar may help protect your dog from reaching their injury or wound. Some dogs may be able to reach the wounds on their legs, paws and tail. The protective inflatable collar is not 100% guaranteed to prevent punctures, chewing or biting that can cause the tube to deflate. Monitor your dog when wearing the protective inflatable collar. **CAUTION: STRONG VELCRO.** Avoid contact with clothing and fabrics that would catch on the hooks.



PROTECTIVE COLLAR SIZE CHART

COLLAR SIZE	SIZE OF NECK (CIRCUMFERENCE)
Extra Small	Up to 6 inches Up to 15.2 cm
Small	6 inches – 10 inches 15.2 cm – 25.4 cm
Medium	10 inches – 13 inches 25.4 cm – 33.0 cm
Large	13 inches – 18 inches 33.0 cm – 45.7 cm
Extra Large	18 inches and up 45.7 cm and up

PROTECTIVE COLLAR

Design features a security strap with secure fit velcro closure and durable fabric which allows dogs to eat and drink and is easier on your dog's neck than a cone

INFLATABLE COLLAR HELPS PREVENT DOGS FROM REACHING INJURIES, RASHES OR POST SURGERY WOUNDS

PROTECTIVE COLLAR

INFLATABLE COLLAR HELPS PREVENT DOGS FROM REACHING INJURIES, RASHES OR POST SURGERY WOUNDS

SIZING: To ensure proper fit, measure the smallest part of your dog's neck. Size up if your dog is between sizes or has long limbs.
WASHING INSTRUCTIONS: Hand or machine washable in warm water, mild detergent and dry flat.

**EXTRA SMALL
UP TO 6 INCHES**

FINISH LINE PETS
a division of FLP, LLC
TEMPE, AZ 85282
FLP, LLC ©2017 / vetworthy.com
(480) 436-0823 / MADE IN CHINA
12201003



**XS EXTRA SMALL
(UP TO 6 INCHES)**

**XS EXTRA SMALL
(UP TO 6 INCHES)**

Size: 3"x3.125"x7"
colors: 4 color process + pms