



PILL GUN



Great for small pets

**EASILY DISPENSE
CAPSULES
& TABLETS
TO PETS**

1 PILL GUN


**OUR VET WORTHY® PILL GUN
HELPS TO EASILY DISPENSE
CAPSULES AND TABLETS TO PETS**

Our pill gun is designed to help administer oral capsules or tablets to dogs, cats or small pets. Always ensure fresh drinking water is available to your pet after administration.



DIRECTIONS:

1. Remove the paper collar. Remove the plunger. Insert the pill into the barrel. Replace the plunger. Push the plunger into the opening of the barrel tip.
2. Place index and middle fingers under the bars of the pill gun. Adjust as necessary for full control.
3. Open your pet's mouth with one hand over the top of the head and muzzle. Apply pressure with your fingers on each side of the the jaw hinge. Insert the pill gun over the tongue and to the back of the mouth. Quickly push the plunger to administer. Rinse with water and air dry. Pull the plunger out when not in use.

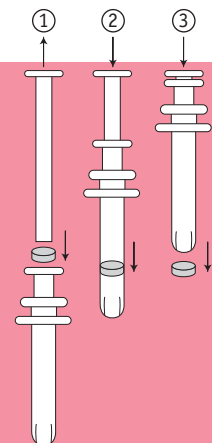

CAUTION: FOR ANIMAL USE ONLY. NOT INTENDED FOR HUMAN USE.

Keep out of the reach of children and animals. This product is not a toy. Inspect product regularly and replace if any part becomes worn, loose or detached. Do not use if product appears to be tampered with or broken.

FINISH LINE PETS
a division of FLP, LLC, TEMPE, AZ 85282
FLP LLC ©2017 / vetworthy.com
(480) 436-0823 / **MADE IN CHINA**
12201825



COLORS:



LABEL DIE: 3.75" x 8.25"

: Blister

COLORS:



LABEL DIE: 3.75" x 8.25"

: Blister

3.75w x 8.25h
front: 4/color process + pms
back: black only