

**vet
worthy®
DROPPERS**



Great for
ALL pets

FOR EASIER
ADMINISTRATION
OF ORAL LIQUIDS

2 DROPPERS



LABEL DIE: 3" x 6"

Back (2 sided adhesive Sticker)


**vet
worthy®
DROPPERS**

OUR VET WORTHY® DROPPER
HELPS ADMINISTER ORAL
LIQUIDS TO YOUR PETS



DIRECTIONS:

1. To fill with liquid, insert the dropper tip into the liquid, squeeze the bulb and release.

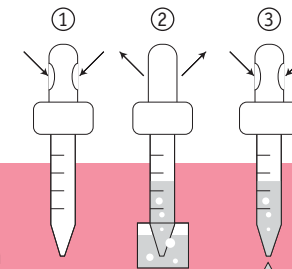

2. Liquid will be sucked into the dropper.

3. Open your pet's mouth with one hand over the top of the head and muzzle. Apply pressure with your fingers on each side of the jaw hinge. Insert the dropper over the tongue and to the back of the mouth. Slowly squeeze the bulb to administer. Rinse with water and air dry. **To avoid inhaling liquid, keep pet's head level. DO NOT tilt head backward.** Feed your pet in an upright position.

CAUTION: FOR ANIMAL USE ONLY. NOT INTENDED FOR HUMAN USE.

Keep out of the reach of children and animals. This product is not a toy. Inspect product regularly and replace if any part becomes worn, loose or detached. Do not use if product appears to be tampered with or broken.

FINISH LINE PETS
a division of FLP, LLC, TEMPE, AZ 85282
FLP, LLC ©2017 / vetworthy.com
(480) 436-0823 / MADE IN CHINA
12201805



COLORS:



LABEL DIE: 3.75 x 8.25

3.75w x 8.25h
front: 4/color process + pms
back: black only